



# Bedpan Washer Disinfectors

For hospitals, nursing  
homes and private clinics

**KEN** HYGIENE SYSTEMS®



# Intelligent Quality

- the future of effective disinfection





# Bedpan Washer Disinfectors

## **Focus on Ergonomics and Working Environment**

The KEN Bedpan Washer Disinfectors are front operated machines which secures the user an ergonomically correct working posture. Furthermore, the bedpan can be placed directly on the door, to be automatically emptied by door closing. Splashes and odor nuisances are avoided. The door makes it applicable as a slop sink.

## **Improve the working day with easy Operation**

The machines are front opened by a simple push button system and have – in standard execution – 5-7 wash programs that can be selected according to the character of the task and dirt. Build-in computer control ensures a homogeneous washing and disinfection, and the integrated water tank and pump makes the KEN Bedpan Washer Disinfectors independent of fluctuations of the external water pressures. The rotating wash heads ensure that all dirt is removed. In case of irregularities during the program cycle, the computer-surveillance registers and reports the error in the form of a code on the front panel.

# KEN ECOLINE BWD 731



KEN TOUCH DISPLAY

## Economy

- Lowest operational costs  
water, electricity and detergent

## Safety

- Self disinfecting
- Built in condensor
- Compliance with EN ISO 15883-1-3
- Reliable washing and disinfection

## Reliability

- Long stable and continued operation  
records more than 25 years of experience

## Useability

- Individual and flexible programming
- Freestanding, wall mounted or  
underbench
- Compact small footprint

## Servicability

- NetCom for ethernet connection
- Easy access





**1: 2350327 Bedpan holder Type: Nordic, German, Dutch.**

This holder is suitable for 1 bedpan or 2 urinal bottles at a time.

2: KEN BWD 731 mounted with a Nordic, German or Dutch bedpan holder with 2 urinal bottles.



3: **8300421 Utensil basket.** KEN BWD 731 has an option for an utensils basket. This basket will hold cups, kidney bowls, vases, scissors and all items that need to be washed and disinfected.

4: **2350321 Bedpan holder Type: French.** The BWD range can be supplied with a suitable holder for any type of bedpans.



5: Many of the holders will also hold urine measure glasses, buckets or other items that need to be washed and disinfected.

6: **8200402 Foot pedal for hand free operation.** To ensure easy and safe handling for the nurses our bedpan washer disinfectors can be supplied with foot pedal for hand free operation.

# KEN ECOLINE BWD 733



KEN TOUCH DISPLAY

## Economy

- Lowest operational costs  
water, electricity and detergent

## Safety

- Unique washing system
- Self disinfecting
- Built in condensor
- Compliance with EN ISO 15883-1-3
- Reliable washing and disinfection

## Reliability

- Long stable and continued operation  
records more than 25 years of experience

## Useability

- Individual and flexible programming
- Special suction or urine bag holders
- Large washing chamber offers great possibilities
- Small footprint

## Servicability

- NetCom for ethernet connection
- Easy access





**1: 2331805 Bedpan holder Type: Nordic, German, Dutch.** This holder will hold 1 bedpan and 2 urinal bottles. The KEN Ecoline BWD 733 is designed with an unique washing system, to ensure optimal hygiene.

**2: 2338000 English Bedpan holder for 1 bedpan and 2 urinal bottles.**

**3: 2338000 English slipper pan small and 2 urinal bottles.**

**4: PS-11045 French Bedpan and 2 urinal bottles.**

**5: 2330838 Bedpan holder. Various types for commodes.**

**6: PS-12073 Austrian Bedpan holder – for 1 Bedpan or 2 urinal bottles.**



# KEN ECOLINE BWD 736/738



KEN TOUCH DISPLAY

## Economy

- Lowest energy consumption in water, energy and detergent

## Useability

- User-friendly design for components and software
- Quadruple use : bedpans, wash bowls, utensils or a Cytostatica version
- Up to 6 different user specific programs
- Very little footprint (depth 460mm)
- Pass-through model available (BWD 738) for separation of unclean and clean zone
- Reliability for more than 20 years



**1: 2360410 Bedpan holder Type: Nordic, German, Dutch.** The KEN Ecoline BWD 736 can hold 3 round bedpans at a time. 1 fully loaded and 2 with urine. Alternatively you can wash and disinfect 3 urinal bottles and 1 round bedpan in one cycle.

**2: KEN BWD 736** with 3 urinal bottles and 1 Nordic bedpan.

**3: 2360413 Bedpan holder Type German.** Various types for commodes. The holder for German bedpans can also be used for the commode shown above. You will be able to wash and disinfect 1 commode and 3 urinal bottles in one cycle.

**4: 2360411 Bedpan holder Type: English, Australian, American.** Special holders for each country or type of bedpans.

**5: 2360411 English, Australian, American** Bedpan holders with several possibilities.

**6: 2342879: Bedpan holder for 2 French** Bedpans.

**For Hand free operation choose reference 8200402**

## Full Service supplier

Our ambition is to be the preferred partner through development and marketing of innovative hygiene solutions and services within the healthcare sector to ensure our customers the highest standards of reliability, functionality, economy and environment.





A vertical photograph of an elderly woman with short, wavy grey hair, smiling warmly as she looks out of a large window. She is wearing a white bathrobe and is seated in a wheelchair. A hand from another person, wearing a grey long-sleeved shirt, is gently resting on her right shoulder. The background outside the window is bright and out of focus, showing hints of greenery and a light sky. The overall mood is peaceful and supportive.





Malone Hospital Services Pty Ltd  
24 Tathra Street  
West Gosford 2250, NSW  
+61 (02) 4323 9731  
sales@malones.com.au  
www.malones.com.au

## **KEN HYGIENE SYSTEMS A/S**

Bøgebjergvej 60  
DK-5672 Broby  
Tlf. +45 62 63 10 91  
Fax +45 62 63 16 07  
ken@ken.dk  
www.ken.dk